


General Preparation for CELL-VU® 20µ Disposable Counting Chamber with Separate Coverslip

- Mix the sample thoroughly just before pipetting.
- Pipette one drop (approximately 4 microliters *) of undiluted specimen on the CELL-VU® slide. * Please note that the CELL-VU® Counting Chamber uses a smaller sample than the Microcell® or Standard Count® (Leja)
- Place the specimen at the extreme edge of one of the sampling areas.
- Gently lower the cover glass over the specimen so that the edge of the cover glass just covers the sample.
- Slide the cover glass into position. This will eliminate air bubbles from the counting area.
- If doing a manual count: Count all motile and non-motile sperm within 10 small boxes within the reticle grid. Divide this total number by 2.

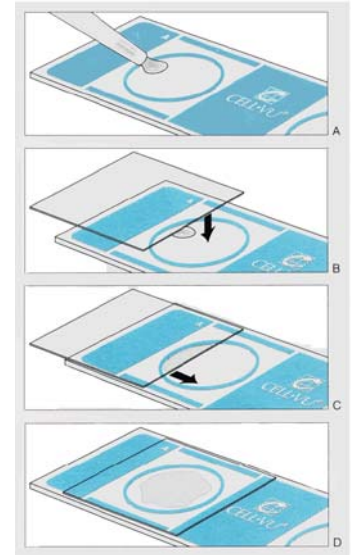

This result is the concentrations of sperm in millions/ml.

$$\% \text{ motility} = \frac{\text{number of motile sperm}}{\text{total concentration}} \times 100$$

- Two tests can be performed using one CELL-VU® slide.

Suggestions:

- For increased accuracy, count all sperm within the grid (100 boxes). Multiply the count by 50,000 to obtain the total concentration of sperm/ml.
- Sperm can be immobilized by placing a small amount of sample in an appropriate container and then immersing it in hot water for several minutes.
- If diluted specimen is used, follow the instructions and multiply by the dilution factor.

Fertility Technology Resources, Inc.
www.fertilitystuff.com

phone: (800) 533-5113 or (770) 641-9487
fax: (866) 641-1560 or (770) 641-1560